
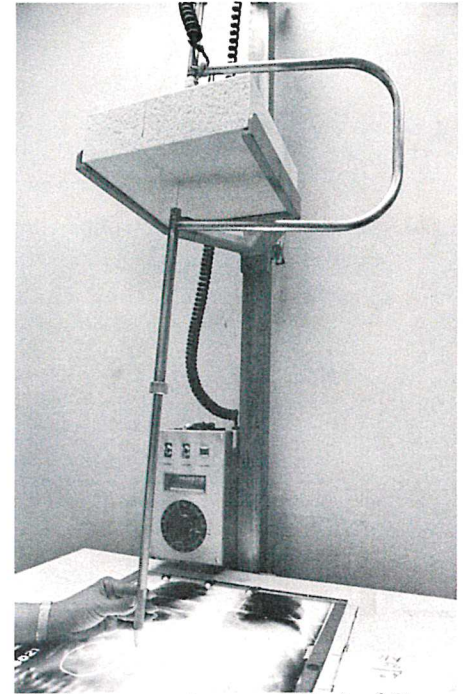


# Cerro will assist you from the purchasing of Cerrobend to locating other required equipment.

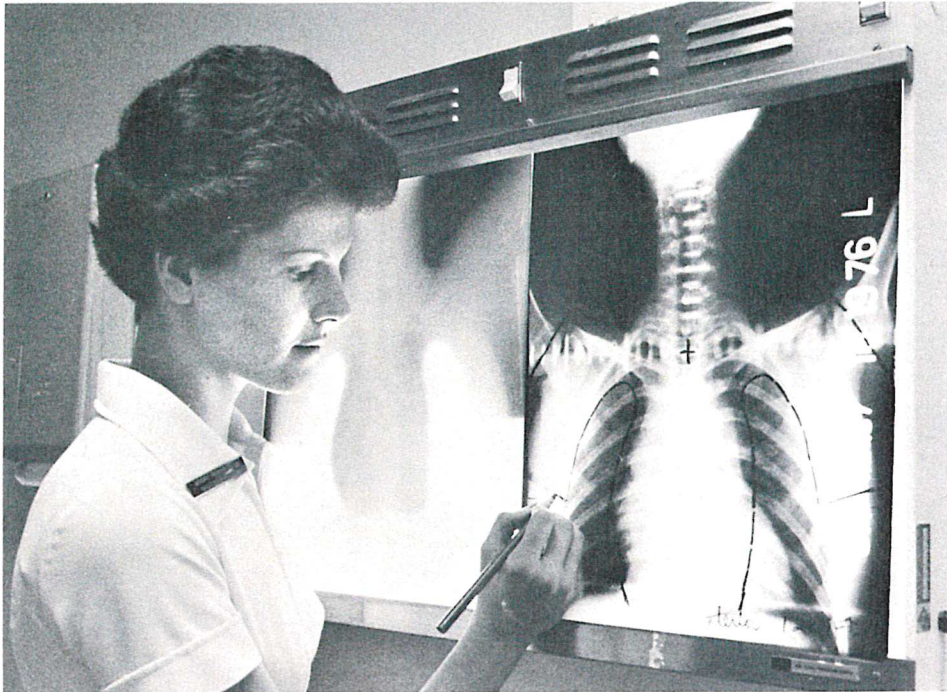

A therapist is outlining on the patient's X-ray the areas to be protected by the

mantles made of Cerrobend.

## Molding Procedure

After simulation tests are completed, and diagnostic X-rays of the area to be protected are outlined, the procedure for preparing a Cerrobend mantle block can begin. While tracing the outline on

the X-ray of the area requiring shielding, a hot wire cutter is duplicating this pattern in a Styrofoam block providing exact angles for maximum shielding of the radiation beam. With this accuracy and ease of cutting, molds can be quick-

The hot wire cutter cuts a block of Styrofoam (top) to the exact outline of the X-ray being traced (bottom).

ly made and prepared for casting. All paths of escape are sealed off with masking tape, leaving one side open for casting.

**Please have your representative contact me.**

- We wish to purchase Cerrobend
- Locating manufacturers of the required equipment

Name \_\_\_\_\_

Title \_\_\_\_\_

Hospital \_\_\_\_\_

Address \_\_\_\_\_

City \_\_\_\_\_ State \_\_\_\_\_ Zip \_\_\_\_\_

Phone \_\_\_\_\_

Type of method presently being used \_\_\_\_\_